E.M.G Disposable Concentric Needle Electrode

Instruction for use

Spes Medica S.r.l.
Via Europa, Zona Industriale 84091
Battipaglia, Salerno-Italia
Tel: +39 010 390343
Fax: +39 010 3072345
web: www.spesmedica.com

	The device complies with the EC directive 93/42/EEC with amendments up to 2007/47/EEC for Medical devices.
	注册证号码: 国械注进 20182211759
US: Rx Only	Caution: In the United state, Federal law restricts this device to sale by or on the order of a physician practitioner.
	Warnings and Cautions associated with this device.
	Single Use Only
	Expire by
	Read Instructions for Use
	Ethylene Oxide Sterilization
	Do not use if package is damaged
	Latex Free

ENGLISH

Disposable bipolar needle electrode Instruction for use

INTENDED USE

Recording muscle activity for Electromyography (EMG) applications.
For single patient only

DIRECTION FOR USE

- Do not use the selected needle if the pouch is damaged;
- Open the pouch and take the needle out of the sterile pouch;
- Connect the needle to the cable needle holder;
- Remove the needle cap;
- Insert the needle into the area to be tested;
- Do not insert the needle up to hub;
- If the needle bends before, during or after insertion, do not straighten or re-insert. Discard and replace with a new needles;
- After use, dispose the needle according to the national laws and regulations or alternatively put the needle in approved biohazard sharps containers.

CAUTION

- The bipolar needle electrode is for single patient only.
- This single-use product is not designed for re-use. Attempts to clean and re-use this single-use device expose patients and operators to a risk of cross contamination.
- Do not re-sterilize.

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