Stimulating and Recording Electrodes

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ISP61_Elettrodi per stim e rec_Ed.0725

Stimulating and recording electrodes
Intended use
The electrodes are intended use for stimulation and recording. Specially use for EMG and NCS.

Instructions for use
① Clean the patient's skin with a damp cotton ball;
② Put the electrode in the select area;
③ Connect the electrode to the equipment for stimulation or recording.

Directions for cleaning
Clean the electrode after each use with a damp cloth and mild detergent. Do not immerse in cleaning solution. Do not use automatic cleaning tools.

Wipe Disinfection
Wipe disinfection with products used for surface disinfection (noncritical item).

Waste Disposal
Waste coming from hospitals must be disposed of in accordance with regulations in force.

	Complies with the Regulation (EU) 2017/745
	Swiss authorized representative
	Medical Device
	Catalogue number
	Lot number
	Unique device identifier
	Consult instruction for use
	Warnings and Cautions associated with this device.
	Not Sterile
	Do not use if package is damaged
	Manufacturer

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