Recording and Stimulating
for Central Nervous
System electrodes

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Web: www.spesmedica.com

ENGLISH

Recording and Stimulating for Central
Nervous System electrodes

Intended use

The electrodes are used for electrical stimulation and recording of the central nervous system signals for intraoperative monitoring, brain mapping and location of epileptogenic foci. The electrodes are multipole arrays of electrodes that are positioned on the cerebral cortex and spine.

Instructions for use

 The surgeon is the best advised for electrode positioning. He will choose at its discretion the best procedure according to clinical and anatomical needs. The next steps are just general guidelines as an example.

The **strip electrode** is inserted into the burr hole or craniotomy site into the subdural space. Multiple strips may be placed through a single burr hole; surgeon will define the best procedure according to clinical need for the insertion. The strip electrode should slide easily on brain surface and it should not be forced.

The **array electrode** is positioned on exposed surface done by craniotomy. The electrode is place on the cortical surface with a direct visual control.

After electrodes placement, the dura mater is closed by absorbable sutures. The electrode exits at the hole or through the bony flap. Take care about cables wiped cleaning.

Take care that the electrode contacts are outside the body exposed or without protection.

At the end of the monitoring or mapping remove the electrodes surgically. Do not take off the electrode pulling the wire.

Warning

The electrodes are provided sterile and can be used for a single patient only. Do not re-sterilize. Do not reuse. Not for implantation (more than 30 days). Do not use if packaging is damaged.

The electrode is not recommended for continuous stimulation. Following surgery, patients with electrodes should be closely monitored.

Disconnect from monitoring equipment during defibrillation. Do not pull the cables with force, possible connection losses. Handle the electrode with extreme care to prevent any damage.

Stimulation should be only applied to support brain mapping. The safety of the electrode in MR is unknown. Make sure to get the total contact between the electrodes and the SNC to avoid an increasing of current intensity.

Waste Disposal

Waste coming from hospitals must be disposed of in accordance with regulations in force.

	Complies with the EC directive 93/42/EEC
	Catalogue number
	Lot number
	Caution
	Not Re-sterilize
	Do not reuse
	Do not open if the package is damaged
	Ethylene oxide sterilization
	Expiration date
	Consult instruction for use
	Manufacturer

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